

Jorvet™

g-Pet™
PLUS+



The new and improved veterinary glucometer

Advantages of the new meter:

- Improved FAD-GDH enzyme on test strip that assures accurate results. This is the preferred measurement method
- Smaller sample volume than previous meters
- Ejects strip at the touch of a button
- Single code strip for cats, dogs, and horses
- Developing - Option of voice instructions and Bluetooth connectivity



Additional key features:

- Calibrated specifically for cats, dogs & horses
- Biosensor auto code technology eliminates need to calibrate strips
- Easy to use
- Vets can train owners to home test
- Small sample (0.7µl) of whole blood is required
- Large LCD screen
- Accurate results displayed within 5 seconds
- Reportable range 20-600mg/dl
- Stores up to 500 results
- Adjustable alert range alarms
- Has wider HCT range - The HCT range for accurate glucose results is 20-60%

GLUCOSE
Veterinary Monitoring System

Why use the gPet PLUS veterinary glucose meter rather than a human glucose meter?

- Human glucometers are NOT accurate when measuring glucose levels in animals.
- Red blood cell size and the distribution of glucose within the blood differ from human to animals.
- Humans have a higher hematocrit compared to cats, dogs, and horses, as well as larger red blood cells.
- A portion of glucose is contained within the red blood cells and the remainder in the surrounding plasma. This differs from human blood.
- The distribution of glucose within plasma is proportionally much lower in humans compared to animals, as a larger portion of glucose is contained within the red blood cells.
- Subsequently, these differences in glucose distribution have a bearing on the glucose result when testing animals with a meter designed for human use.

Other thoughts/considerations

- By providing the glucose meter and test strips your client has the sense of how well the blood glucose is being tested.
- Interaction with the client is increased verses the client discussing the glucose meter use with a non-veterinary pharmacist.



J1376

Kit includes:

- gPet PLUS meter
- AAA battery x 2
- Test strips (35)
- Code strip
- Control solution
- Carry case
- J1376S Test Strips (50pk)
- User guide
- Test strip insert
- Control solution insert
- Quick reference guide
- Lancing device
- Lancet x 10

Testing Made Easy!

Insert the test strip
(Meter will turn on automatically)

Step 1



Apply Blood

Step 2



Results in 5 seconds

Step 3



Jorgensen Laboratories

www.JorVet.com Info@JorVet.com (800) 525-5614